Speedex Universal Activator

Definition

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Definition

Speedex Universal Activator is a silicone-based impression material.

Material type

Polysiloxane, condensation-type silicone elastomer. Universal activator: green

Indications

Universal Activator for Speedex putty (tray-material), Speedex light body (washmaterial) and Speedex medium (washmaterial).

Clinical Time (Traymaterial)

Clinical Time (Washmaterial)

Contra-indications

Provided that the product is used as directed, there are no known contra-indications.

In case of contact of the catalyst with the eyes, wash out with plenty of water and then consult the ophthalmologist.

Usage

Only use together with Speedex putty / light body / medium. Please carefully read instructions for use of base materials.

Shelf life and storage

Speedex fulfills the intended purpose at least until the expiry date, which is displayed on the containers, in well sealed containers at temperatures 15–23 °C / 59–73 °F and 50 % relative air humidity. Impressions should be stored at normal room temperature, avoid exposure to heat and sun. All C – silicone activators are moisture sensitive. Please ensure cap is replaced promptly after use to prevent crystallisation.

Caution

Federal law restricts this device to sale by or on the order of a dentist.

Marking

The expiry date and number [LOT] are shown on the package and cartridge.

Date of issue

01-2014

Only supplied to dentists and dental laboratories or upon their instructions.

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