---

# "EGEO EG-18L" dry heat sterilizer USER 'S MANUAL

## **Installation - use - maintenance**

### **Note:**

- Before unpacking the "Eg-18L dry heat sterilizer", ensure the box has not been damaged or knocked during transportation.
- Read carefully the instructions before operation.

### **Installation**

- Plug the sterilizer in a 220/110 volt. socket. (Do not change either the wire or the plug. Do not employ any adapters which might obstruct the earth connection).

The new line of temperature controllers has been developed with the newest digital microprocessors available. These multifunction controllers allow users an easy handling of the equipment as well as a simple visible interface.

### **Operation:**

The controller operates as a conventional PID controller.

### **Programming:**

Simple access to temperature programming according to the user needs.

### **Program selection:**

Users enter the program selection menu by pressing the "P" key once. The device will show the operating programme. After pressing the "P" key again, the next programme will be shown. The display will show the number of the programme as well as the corresponding temperature and time.

Programme 1 Manual

Programme 2 160 °C – 120min.

Programme 3 165 °C – 100min.

Programme 4 170 °C – 90min.

Programme 5 180 °C – 60min.

Users select the programme shown on display by pressing "ACCEPTAR" key.

After selecting programme 1 MANUAL and pressing "ACCEPTAR" key, users are able to set temperature through the "UP & DOWN" keys. All data selected will be validated by pressing the "ACCEPT" key again.

The temperature maximum limit is 200 °C.

The time maximum limit for manual temperature is 150 min.

Once the selected temperature level was achieved, selected time countdown will start.

## **Technical features**

### **Technical characteristics**

	HEIGHT	WIDTH	DEPTH
Outer walls	31 cm.	45 cm.	30 cm.
Inner walls	22 cm.	32 cm.	25 cm.
4 trays (Size: 31cm x 19 cm)			
Weight: 15 kg.			
Capacity: 18 lt.			
Voltage: 220/110 volts, 50/60 cycles.			
Power: 400 W			
Consumption: 2,00 AMP.			
Warranty: 1 year			

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