CE  
0499

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**PROFESSIONAL USE ONLY**

**Composition:**

The powder contains: Strontium Aluminum Silicate, Dehydrated Polyacrylic Acid and Iron Oxide.

The liquid contains: Polyacrylic Acid, Tartaric Acid and Distilled Water.

**VITRO LINER** is chemically activated acid/base glass ionomer cement. It is radiopaque due to the strontium action and is easily manipulated. It presents the features that are common to other glass ionomers, i.e., biocompatibility, high chemical adhesion, low solubility, adhesion to wet surfaces and continuous fluoride release.

**Indications:**

**VITRO LINER is recommended for lining. It works as a base for composites, amalgam, ceramic and metal restorations. It can also be used as a sealant for pit and fissure.** It presents high radiopacity and fast hardening.

**Presentations:**

Box containing: 1 bottle of **VITRO LINER** powder (5 g) + 1 bottle of **VITRO LINER Liquid (4 mL)** + 1 measuring spoon.

Box containing: 1 bottle of **VITRO LINER** powder (10 g) + 1 bottle of **VITRO LINER Liquid (8 mL)** + 1 measuring spoon.

Box containing: 1 bottle of **VITRO LINER powder** (10 g) + 1 measuring spoon.

Box containing: 1 bottle of **VITRO LINER liquid** (8 mL).

Box containing: 1 bottle of **VITRO LINER liquid** (10 mL).

**Storage and Conservation:**

**VITRO LINER** must be stored between 15 °C and 30 °C. Keep the bottles of powder and liquid tightly closed after use. Do not expose to excessive heat. Under these conditions, the product is valid for up to 3 years as of the manufacturing date. Do not use this product if shelf life had expired.

**Instructions for Use:**

The use of gloves is recommended during the procedure.

**Attention:** The patient must be informed on the risk to occur sensitivity in the place of the procedure. This sensitization can be minimized through the use of proper isolation (rubber dam) and pulp protection.

To obtain the maximum adhesion, the dentin should be conditioned with **VITRO CONDICIONADOR** for 10 seconds. Wash enamel and dentin with water and dry without dehydrating. A film of moisture, as occurs frequently in vital teeth, can be incorporated into the cement.

**Dosage**

Gently shake the bottle, to release the powder. Fill the measuring spoon with powder and level it in the stopper.

The proportion used should be of 1 measuring spoon of the powder to 2 drops of the liquid. The liquid should be dispensed while holding the bottle in a vertical position. Any crystals present in the neck should be removed, to assure that there are not air bubbles in the dispensed drop. Close tightly the bottles before mixing.

**Mixture**

You can use a metal spatula, plastic spatula is not necessary. The mixture should be made on a mix pad or a glass plate for 20 seconds. The powder should be added to the liquid in two stages, being the second, after the first complete homogenization. The powder and the liquid should be dosed in order to obtain a creamy consistency. The mixture should be done in a temperature between 20-30 °C.

Setting time is valid for the specific temperature and humidity condition described. Higher temperatures will **reduce** working time and lower temperatures will **increase** working time.

Mixing Time (21-23°C, 50-52% UR)	20 sec
Working Time (21-23°C, 50-52% UR)	> 2 min 30 sec
Net Setting Time (36-38°C, =90% UR)	3 to 4 min

**ATTENTION:** Glass plate could be cooler if a longer working time is desired.

**Application**

During application and light-curing, you should avoid contact with water or saliva. In cases where the residual dentin layer is equal or inferior than 0,5mm or in the presence of exposed pulp. It is recommended to apply calcium hydroxide cement before **VITRO LINER** application.

The product should be applied after isolation and drying of the cavity. Apply **VITRO LINER** in the cavity with a syringe Centrix to prevent the formation of air bubbles inside the material. Wait for the cure of the material before continuing with the restoration.

In composite resin restorations:

- The application of **VITRO LINER** should be restricted to the deepest point of the cavity, thus allowing the adhesive to form a hybrid layer.
- Avoid the contact between the phosphoric acid with the **VITRO LINER**. However, if the contact happens, it is not necessary to remove the product, continue with the adhesive system.

**Product wastes must not be reused. They must be disposed as the local law in force.**

**Precaution:**

Avoid the use of **VITRO LINER** in patients with previous history of allergy to glass ionomer cements. In case of allergy, you must discontinue the use immediately and advise the patient to see a doctor. The liquid and the prepared cement can be corrosive. Do not allow contact with soft tissues or the skin. In case of contact, you should immediately remove the material with hydrophilic cotton soaked with water and rinse thoroughly with water. Avoid contact with the eyes; in case of accidental contact, wash the eyes immediately with water and see a doctor. Do not mix the powder or liquid which are components of this product with any other glass ionomer-based product. This product must be used only by surgeon dentists and for the intended use described in the Instructions for Use. Avoid contact with eugenol-based products, since they can delay the action of glass ionomers.

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DFL Indústria e Comércio S.A.